



movita

Nursing home beds





movita nursing home beds Economical for nursing institutions, comfortable for residents

The nursing care market is subject to many influencing factors and trends, such as the increasing number of people in need of acute care or are affected by dementia. This has operators and staff of residential homes face increasing qualitative and economic challenges. In the light of the increasing shortage of qualified nursing staff, this applies all the more. The sector needs concepts and products that make nursing tasks easier and convince on an economic level. The movita nursing home beds combine functional elements with cost effectiveness and homeliness.

Factors influencing the nursing care market and trends*


	Factor	Trend
lursing are	Demographics	Change in the demographic str of society, rise in life expectanc increasing risks (multimorbidit decreasing support potential in
ector	Public authorities	Change of political guidelines, predictability of politics, fewer p supply-side subsidies, etc.
	Customer	Changing demand patterns



Living concept Ocean

movita | Design T10 | Dekor Lindberg Eiche

A color combination like a day on the beach gives a feeling of peace and harmony. Intuitively we close our eyes, listen to the waves, enjoy the fresh sea air and a vacation feeling of well-being quickly sets in. You feel at ease in a room like this. This concept is just a suggestion – we will happily give you individual advice.

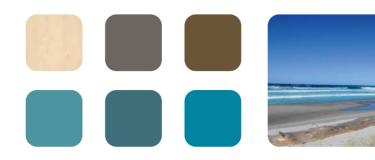



movita Portfolio

The mobile one



The homely one



The narrow one

The one easy to disassemble





movita	movita casa	movita 2-s	movita 2-d	
Movability Movable at every bed height Castors	Movable at every bed height	Movable at every bed height	Movable at every bed height	At first glance, the movi The technical paramete once you take a closer l reassembled by one pe
100 mm easy-roll castors or 125 mm design castors	50 mm double castors	100 mm easy-roll castors or 125 mm design castors	100 mm easy-roll castors or 125 mm design castors	components weighs mo requirements. The movita 2-d is used without compromising c
Braking capacity Individual or central castor brake system with one directional castor, operated with foot levers on both sides	Castor braking system for each axle, operated with foot levers on both sides	Individual or central castor brake system with one directional castor, operated with foot levers on both sides	Individual castor braking	
Height adjustment Approx. 36/38 - 78/80 cm	Approx. 37 - 80 cm	Approx. 36/38 - 78/80 cm	Approx. 36/38 - 78/80 cm	
		Special Feature	Special Feature	

Lying surface width 80 cm

Can be completely disassembled



ovita 2-d cannot be distinguished from a standard movita bed. eters of both beds are also identical. You can only see the difference er look – the movita 2-d can be completely disassembled and person – quickly and without tools. None of the individual more than 18 kg, complying with all occupational health and safety

ed in home care or nursing homes with narrow doors – ng design, quality or stability.

Benefits at a glance



Protection and SafeFree[®] – the pater

Side guards – ind SafeFree® FLex

Bedsore preven Patient surfaces – erg

Flexible & time-3 bed lengths that car

Improving and r SafeLift | Revolutiona

Intuitive operati

Manual mobilisa MobiStick 2

Energy efficience Thanks to innovative

Safety by day ar SafeSense® - the intell

Additional optio

mobility without FDM* ented side guard concept	10 - 11
ndividual according to requirement	S 12 - 13
ntion gonomics & modularity	14
e-saving an be adjusted without tools	15
maintaining mobility ary bed operation for residents	16
tion nd ergonomic	17
sation assistant	18
cy and more user friendly drive concept & power-saving lights	19
and night elligent bed exit sensor system	20
ONS decors Features and accessories	22

Protection and mobility without FDM*

SafeFree[®] the patented side guard concept

The SafeFree® divided side guard system is patented and voluntarily certified. The technology has been tested in practice more than 1,000,000 times. You can lower the space-saving side guards quickly in only 2 seconds and individually adjust them to four heights, depending on the care situation and size of the resident. Obtaining freedom instead of taking it away, in agreement with the Werdenfelser Weg (Werdenfelser care pathway)**.

No protection needed



Lowered, the divided side guards look homely and provide unhindered access to the resident for daily routine nursing tasks. No need to take off and store accessories that are in the way, such as a middle post or fixed side guards.



Soft protection (no FDM*)


For safety at night, it is often sufficient to combine the low position of the bed with the first height level of the head-end side guard. This avoids unnecessary barriers and effectively reduces the risk of falls.



³/₄ protection (no FDM^{*})

Creates a deliberate bed egress gap at the foot end to reduce freedom-depriving measures (FDM) while at the same time maintaining a high safety standard.


More safety due to smaller central gap* < 60 mm

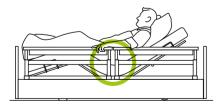

- Targeted compliance with the standard IEC 60601-2-52
- · Safety in working position, i.e. also with raised backrest or leg rest
- · Safety without using an additional central gap protector as accessory for restraint with an abdominal belt in accordance with the BfArM** (German Federal Institute for Drugs and Medical Devices)

All-round access for nursing tasks = with 4 divided side guards

3

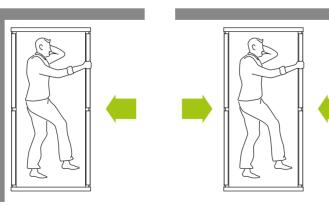

- Flexible use of the bed for standard care, even in limited spaces
- · Flexible access from both sides of the bed in case of increased need for care
- · Flexible use of the bed for bed egress on the left or right side to mobilize the resident
- · Avoiding follow-up costs for upgrading divided side guards

4

Full protection (FDM*) for mattresses up to 18 cm



The highest position provides comprehensive protection of the resident. The distance between the two parts of the side guards is minimal, making them just as safe as a continuous side guard and exceeding the requirements of the standard IEC 60601-2-52. It is not necessary to stock up on accessories to close a central gap when using an abdominal belt for restraint.

Safe mobilization



regardless of body size



Divided side guards are very useful for effective mobilization, as they are easy to grab and clasp. They can be used as a safe standing-up aid and verifiably support the resident's mobility.

10 wissner-bosserhoff | movita * Freedom-depriving measures | ** see "Argumentationshilfen zu unterschiedlichen Bettgittervarianten" (argumentation aid for various side guard types) http://werdenfelser-weg-original.de/medien/

Learn more:



If you use a thicker mattress, you can raise the side guards

by an additional 6 cm.





SafeFree[®] Flex

Side guards as individual as your needs

In addition to permanently mounted side elements, individual and future-oriented SafeFree® Flex offers as much freedom as possible and as much protection as necessary. At the same time SafeFree® Flex retains the homely design which distinguishes all wissner-bosserhoff nursing home beds.

()

No protection needed



Comfort bed, without side guards

1|2

3

Soft protection (no FDM*)

³/₄ protection (no FDM*)



Wall protection with foot-end exit

Safe

Complete protection



- Secure fastening of elements
- · Prevents an accidental loosening or removal of the side elements



Seamless safety

- Gap closure between the side panel and the patient surface
- · Protection against accidents through getting trapped or wedged

Flexible mobilization



- · Optimum length and individually adjustable height of the head-end side guard
- · Perfect hip protection when sleeping as well as comfortable and ergonomic mobilization of the resident



Mobilization aid

only the head end elements

Fewer FDM*

Flex

More economic efficiency



- · Conscious "omission" of side guards
- Investment security for the operator Legal certainty regarding FDM* for the nursing staff

side elements can be attached, because you only pay for the elements that your residents really need.

No technician needed

Quick modification for the changing resident situation - with gap closure without (1) and with a handrail (2) or fully adjustable side guards (3)

Learn more:





4




Full protection (FDM*)

for mattresses up to 18 cm



4 fully adjustable side guards

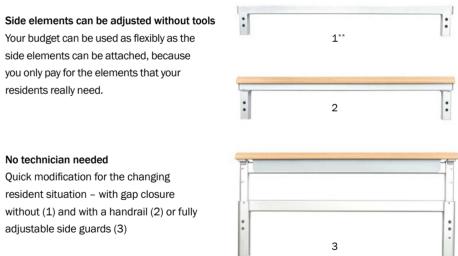

Benefit from a homely design with a continuous side panel, independent of the required mobilization or resident protection situation.

Bedsore prevention

Patient surfaces - ergonomics & modularity

The so-called double retraction has been a component of the wissner-bosserhoff patient surfaces for years. Here, not only the backrest but also the leg rest is moved away from the seat section, so that the surface area is increased. Results of pressure measurements clearly show that the pressure load with the double retraction is considerably lower than with a standard lying surface.

approx. **37.5** % of all residents in German old people's or nursing homes are subject to an increased risk of decubitus

approx. 2.7 m. People suffer in Germany from chronic wounds

like decubitus

Source: Study BVMed 2012

Patient surface modularity

3 different surfaces


Source: Own survey, wissner-bosserhoff 2020

200 cm

Flexible and time-saving


· Aadjusted flexibly and without tools to different bed lengths.

Bed length of 200 cm: The divided side guards offer protection without a central gap > 318 mm* and without a protector

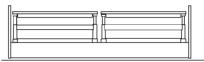

210 cm

Alternative bed length of 210 cm: Protection with longer side guards at the foot end without accessories and without a gap > 318 mm**

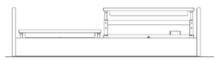

220 cm

Bed extension to 220 cm: is temporarily adjustable with divided side guards for movita - complete protection through foot-end protector (accessory) is guaranteed


<u> </u>		TT (

Our solutions

WIBO patient surface with notable relief



Double retraction = decubitus prevention

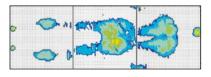


Thermal image of a BiNetic patient surface




Standard patient surface without relief



Single retraction = risk of decubitus

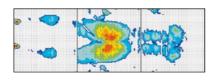


Thermal image of a standard patient surface

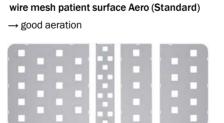

Synthetic mats EasyClean

 \rightarrow easy to clean

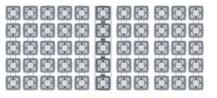

Comfort patient surface with its 55 spring elements → bedding comfort and simple pressure relief


wissner-bosserhoff | movita

3 bed lengths that can be adjusted without tools

· All standard wissner-bosserhoff nursing home beds are equipped with an integrated bed extension.

• The requirements of the standard IEC 60601-2-52 are complied with in each configuration.



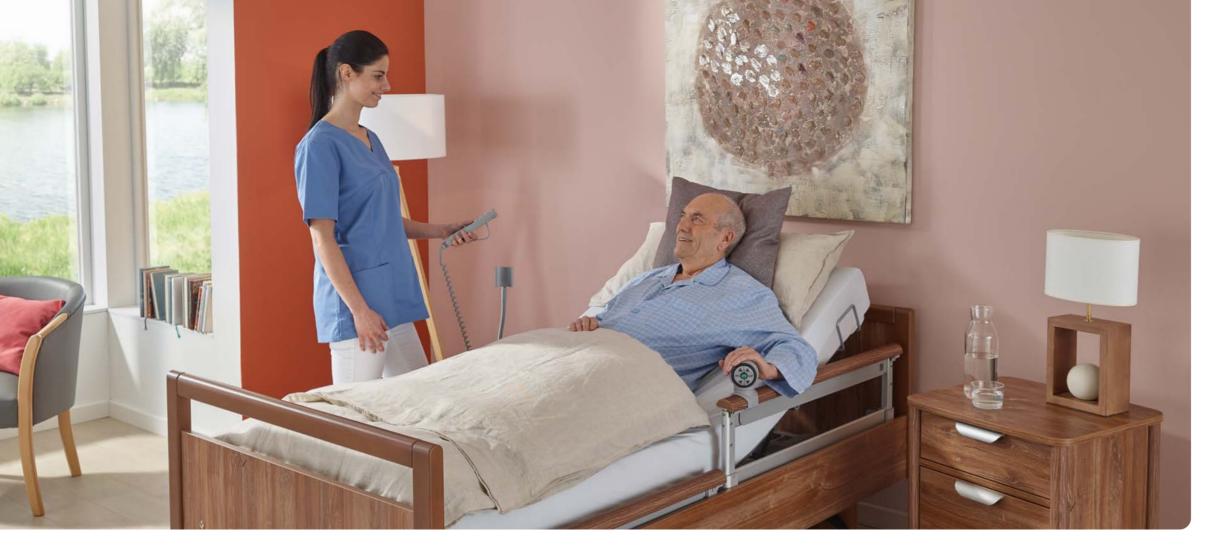


Simple bed operation & safe mobilization

SafeLift | Product benefits at a glance

SafeLift offers the simplest operation so that residents can adjust their bed independently and set it to the best mobilization position for them. At the same time, they can support themselves when standing up on the SafeLift controller. This promotes residents' selfmobilization, and reduces staff calls for bed adjustment.



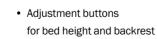

- Large buttons
- · Firm position on a SafeFree® side guard
- GO button
- 360° rotation
- Ergonomic design
- Strong stability



Even more straightforward and ergonomic


Already at first glance, the handset of the movita nursing home beds impresses with its clarity and ergonomic form: fewer buttons and large symbols ensure that operation can be learned intuitively.

The pre-selection function makes the handset single fault safe triggering a function by accidentally pressing a key is not possible.

Advanced Functions



Key for safety levels (magnet-free)

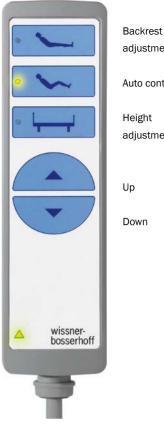

adjustment Auto contour Height adjustment

Down

3 safety levels







Care mode

Locking mode

Resident mode

The locking function is integrated into the back of the handset, meaning that you do not have to look for a control box at the patient surface panel or bend down. The three handset modes can be selected with a non-magnetic key: Care mode: All selection buttons are unlocked. Locking mode: All functions of the handset are locked. Resident mode: The resident can use the upper three buttons.

www.wi-bo.com/de/Ueber-



Manual mobilisation assistant

MobiStick 2

The MobiStick 2 enables clever bed operation for residents. Its ergonomic shape makes it easy and intuitive to use anywhere. Residents can effectively support and move themselves.





Due to its ergonomic shape MobiStick 2 is easy and intuitive to use and universally applicable. The hand knob is rotatable and has a simple locking.



When not in use, it can simply be folded away and is therefore not in the way, but quickly ready for use when needed.

Without tools

Learn more:



Energy efficiency and more user friendly

Thanks to innovative actuator concept & power-saving lights

With its new actuator, movita nursing beds are more user-friendly, more sturdy and more energy-efficient. The integrated LED technology, the energy-saving equipment and the service-friendly design are key requirements for an economic operation of a nursing home.



More user friendly

The new movita is equipped with an LED bedside light connection integrated in the motor. You only need one single cable from the bed to the power supply. The LED light can also be switched on and off using the handset. You can now also watch video tutorials on how to operate the bed by scanning the QR code on the bed.



Video tutorial, also at: www.wi-bo.com/de/Ueberuns/Anleitungsvideos

Comfort LED reading light lymera



Power saving lights The undulating shape of the lampshade, with its partially translucent character, is prominent as a special design element of soluna lights. Equipped with a swivel arm, the light allows the light focus to be precisely positioned close to the object to be illuminated, regardless of whether the light is attached to the bed or the bedside



- Long service life
- · Low energy consumption

cabinet.

- No heat build-up
- Safe operation

Design LED light soluna with wavy lampshade

Economical & energy efficient

Energy efficiency is becoming an increasingly important factor due to steadily rising electricity prices. The new actuator is extremely economical with a very low standby consumption of only 0.9 W. Top cable management ensures less wear. The power cable is interchangeable, which means less storage costs.







Safety by day and night

SafeSense® – the intelligent bed exit sensor system

wissner-bosserhoff, your reliable partner for the reduction of freedom-depriving measures (FDM): With SafeSense[®], wissner-bosserhoff presents an intelligent assistance sensor system, which reliably helps reducing freedom-depriving measures day and night. It is equipped with an automatic night light, which is activated when the resident leaves the bed and is deactivated when the resident is back in bed again. SafeSense[®] is compatible with standard nurse call systems and can be retrofitted to wissner-bosserhoff nursing home beds manufactured in 2002 or later.



Automatic night light

Individual nurse call

• Freely adjustable immediate call from 0 seconds to 30 minutes

· Adaptation to the resident's mobility

- Night light is automatically switched on when the resident leaves the bed
- Night light is switched off again as soon as the resident is back in bed

Integrated or available as retrofit kit*

- Accessory option for current WIBO nursing
 home beds
- Retrofit kit for electrical WIBO nursing home beds manufactured in 2002 or later
- Available with cable or wireless



More safety – less freedomdepriving measures (FDM)

- Fast response time in the
- Safety during night and
- weekend shifts
- Fewer inspections
- Ideal in combination with
- and SafeFree® side guard






wissner-bosserhoff | movita



Design | head / foot sections

Colors | Wood décors*

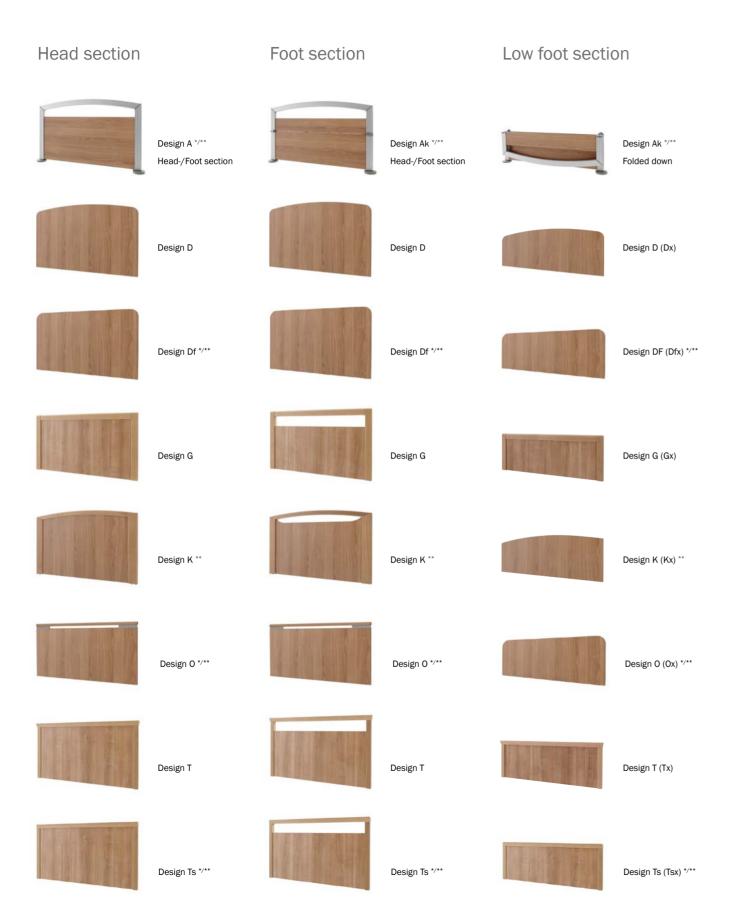



Color combinations for the memoriana room concept*3





Norway maple Salisbury Elm natura



NEW: Norway maple (R27044)*2



Lindberg Oak (R20021)



NEW: Salisbury Elm natural (R37016)*2



Havana Cherry (R42006)



NEW: Okapi Walnut (R30135)*2




Havana Cherry . Okapi Walnut

Furnishings and accessories

10-0766

Dimensions

10-0740 movita 2-s L 208 cm, W 93/95 cm H1 36-38 / H2 78-80 cm

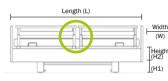



EasyClean

L 208 cm, W 103/108 cm

H1 36-38 / H2 78-80 cm

movita 2-d (only with side guards 09)

Divided side guards (10), without gaps

Comfort

Patient surfaces

Bed extensions

Temporary bed

Options

Vertical

castors

wall stopper

extension divided

10 side guard (standard)

Divided side guards (11), without gaps

movita L 208 cm, W 103/108 cm H1 36-38 / H2 78-80 cm

10-0540

П П





10-0574

movita casa

Continuous 2-part side guard (09) with trigger release


Aero (Standard)

L 208 cm, W 103/106 cm H1 38 / H2 80 cm




Continuous 3-part side guard (09)

Accessories

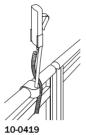

Handset holder for divided side guards 10-0421 Handset holder, short SD-Handset 10-0790 Handset holder, short SafeControl

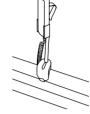

10-0422 Accessories and handset holder for continuous side guards 10-0421 Handset holder, short SD-Handset 10-0790 Handset holder, short SafeControl




10-0634 Mobilizations-Assistant-MobiStick 2






50-0174

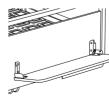

10-0719 Linen holder


for 09 side guards as well as ³/₄ side guards

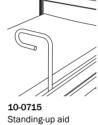



Fall prophylaxis mat with anti-slip W 90 x L 200 cm, height approx. 4.5 cm



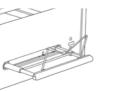

10-0617 Integrated design linen holder Compatible with all beds (except 09 3-part side guards and A-kk design)



Permanent hed

09 side guard (optional)

extension

3D horizontal and vertical wall stopper castors

Permanent bed

extension divided

10 side guard (optional)

CPR release on the backrest



Plug & play use of the handset on one or both sides of the bed

Bed extension

(optional)

with express release

01-010812, 01-010813 Underbed light switchable (not shown)

Protector for bed extension

by 20 cm (optional)

10-0570/10-0580

50-0771





50-0345 Shortened urine bottle basket 50-0164 Urine bag holder

(without illustration)



50-0550 Design LED light soluna without plug power supply



50-0338 Cushioning for 09 two-part side guards

10-0817

):[

Gap closure base (Image similar)

:


10-0818 Gap closure design



10-0819 Side element Flex



50-0677 Comfort LED reading light lymera (with & without power supply) Transparent lampshade

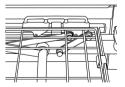

10-0563/10-0533/10-0521 Segufix bracket for thigh-backrest, backrest- for the seat part



10-0548 Head-end wall spacer for movita



10-0587 Head-end wall spacer for movita casa



10-0549 Side-mounted wall spacer for movita/movita casa


For further accessories we recommend our accessory brochure 97-0882.

Accessories | SafeSense[®] & SafeLift

Certified security


0

SafeSense[®] sensor mat 2.1

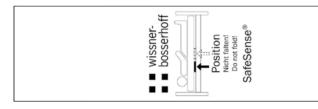

02-000876 SafeSense® sensor mat 2.1 80 x 20 cm

SafeSense[®] connection kit

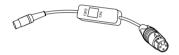

04-1508 Adapter cable with on/off switch and suitable plug for connection to the nurse call system (customer-specific) for the cable and wireless version



10-0812

Digital bed exit box 2

cable or wireless version

04-1608 Adapter cable without on/off switch; with suitable plug for connection to the nurse call system (customer-specific) for the cable and wireless version

Also available

as a set: Sensor mat 2.1

00

пΟ

& bed exit box 2.1 10-0871 Cable version 10-0872 Wireless version

ZERTIFIKAT Name Annual of Street, or other -------------eta (



ISO 9001:2015 ISO 13485:2016

Certificate for side guards



SafeLift bed operation





10-0811 SafeLift Controller






10-0799 SafeLift set







Retaining plate SafeSense® and SafeLift

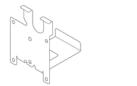



01-011193 Retaining plate for DUO9 motor 01-011194 Retaining plate for Q7 motor

Bedside tables and other accessories

you can find in extra brochures





wissner-bosserhoff | movita

Voluntarily, wissner-bosserhoff has its manufacturing processes and management regularly audited by TÜV Süd. The company is certified in accordance with ISO 9001 for quality management systems, DIN EN ISO 13485 for the manufacture of medical products and ISO 14001 for environmental management.

ISO 14001

www.tuev-sued.de/ms-ze









Members of LINET Group

wissner-bosserhoff GmbH

Hauptstr. 4-6 | 58739 Wickede (Ruhr) | Germany Phone: +49 2377.784-0 | Fax: -163 | info@wi-bo.de | wi-bo.com

